


Oxford®

LONG SEAT SLING

USER GUIDE FOR THE LONG SEAT SLING

The Long Seat Sling is designed for amputees but is also suitable for other patients who have to remain in the sling for an extended period, such as for bed making or bathing.

It is suitable for the lifting amputees, but patients must be individually assessed for suitability by a qualified person. Advice is available from our authorised dealers.

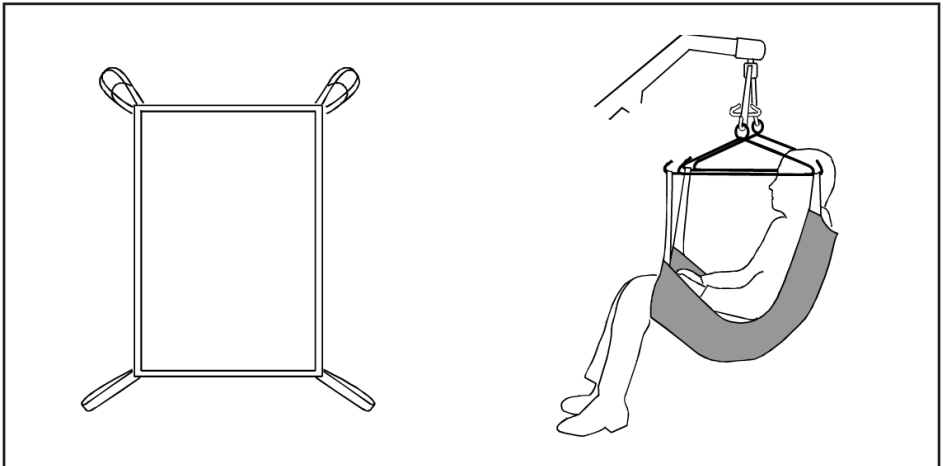

OXFORD RECOMMENDS THE USE OF GENUINE OXFORD PARTS.

Oxford sling and lift products are designed to be compatible with one another. For country specific guidance on sling use and compatibility, please refer to the sling label or contact your local market distributor or Joerns Healthcare.

For the safety of the patient and carer, before using a sling a full risk assessment must be conducted to ensure that the correct sling choice, method of positioning in the sling and procedure for transfer has been determined for the patient.

Check sling and stitching before each use. Using bleached, torn, cut, frayed or broken slings is unsafe and could result in serious injury or death to the patient.

Destroy and discard worn slings. Do not alter slings.

Never leave a patient unattended.




**ADDITIONAL SLINGS ARE AVAILABLE TO SUIT INDIVIDUAL NEEDS.
YOU ARE ADVISED THAT YOU SHOULD ALWAYS SEEK THE ADVICE OF AN
AUTHORISED DEALER BEFORE PURCHASING.**


WE WILL BE PLEASED TO ADVISE YOU OF YOUR NEAREST AUTHORISED DEALER.

HOW TO FIT A LONG SEAT SLING

RECUMBENT POSITION

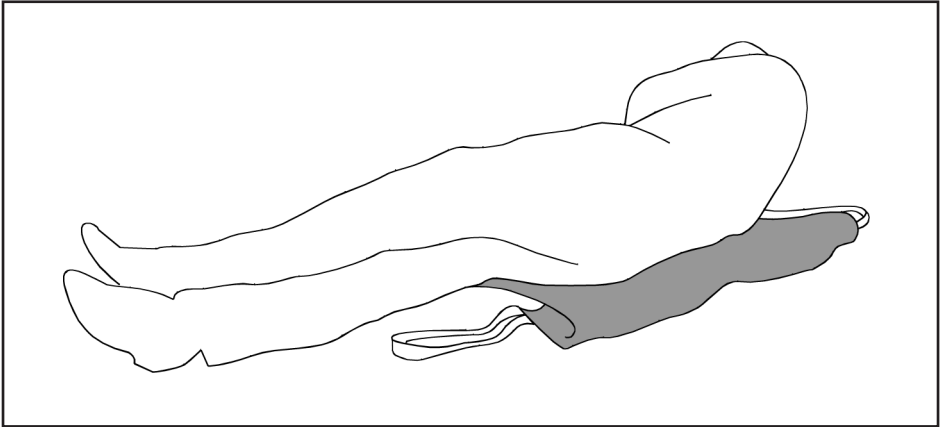


From the recumbent position, roll the patient onto the sling using the draw sheet roll. Ensure the bottom of the sling is just behind the knees.

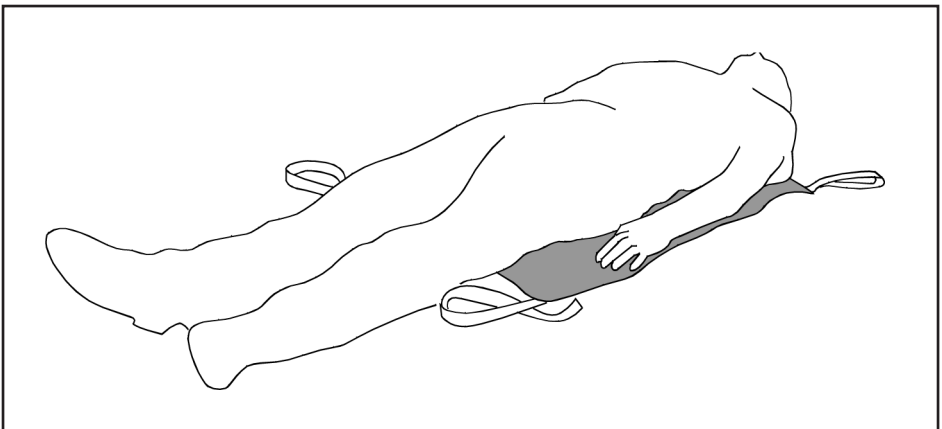


Ensure the sling is flat under the patient. The top of the sling should be at shoulder blade height and level across the top and bottom edges.

HOW TO FIT A LONG SEAT SLING

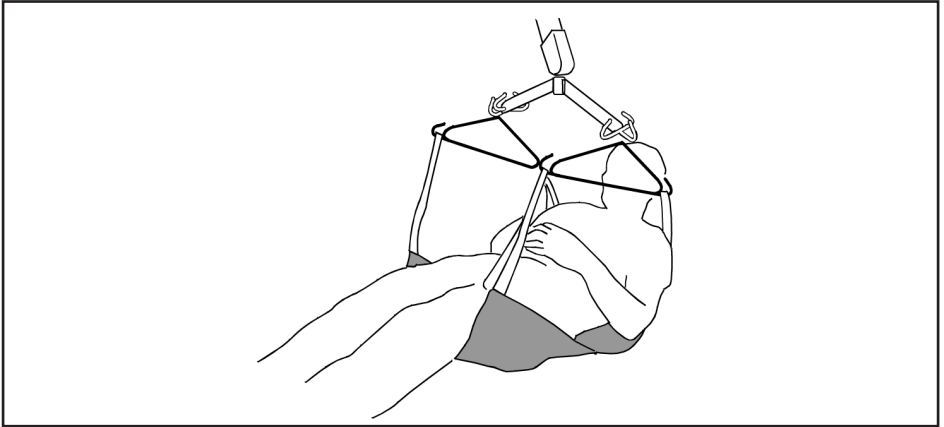

Move the lift into position above the patient. Attach the side suspenders. Keep the side suspenders away from the patient's face. Attach the shoulder strap and leg loops to the respective front and rear hooks on the side suspenders and the centre loop to the centre hook on the spreader bar.

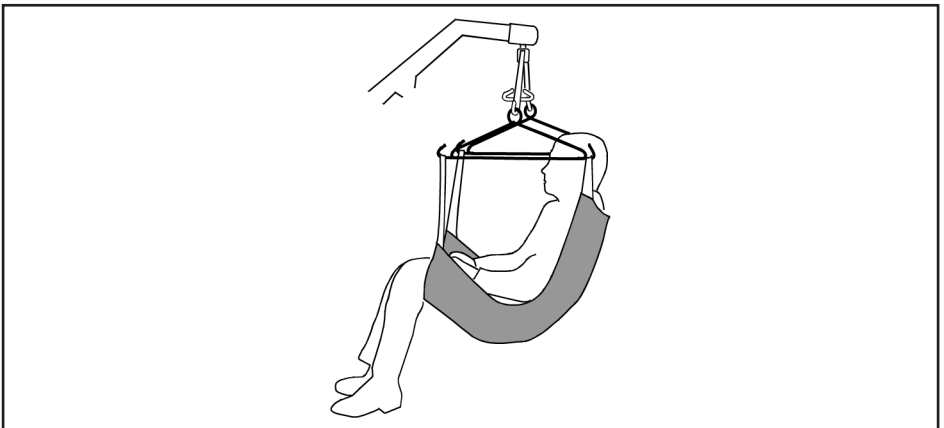

Raise the lift to the required height. NOTE: It may be necessary to support the head while the patient is suspended.


Joerns Healthcare recommend slings are checked regularly and before use for fraying or damage.

LONG SEAT SLING TECHNICAL SPECIFICATIONS

| | |
|-------------------|-----------------|
| STANDARD | BS EN ISO 10535 |
| SAFE WORKING LOAD | 500lbs / 227kg |

SIZING

Each sling is marked with the following colour coded symbols, four of which will be crossed. Your sling is the uncrossed size, i.e. in this example Small (Red).

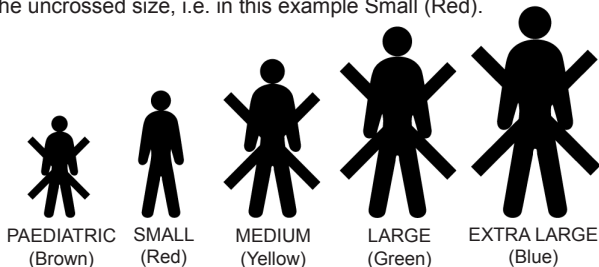

WASHING INSTRUCTIONS

Each sling is marked with the following washing instructions:

DO NOT WASH WITH BLEACH

Bleach will damage material. Slings can suffer damage during washing and drying. Check sling before each use. Bleached, torn, cut, frayed or broken slings are unsafe and could result in serious injury or death to patient. Destroy and discard worn slings. Do not alter slings. Machine wash warm. Cool tumble dry, air dry or dry at very low temperature.



| | | Long Seat (Polyester) | Long Seat (Net) | Long Seat Commode (Poly) | Long Seat Commode (Net) |
|-----------------------|--------------------|--------------------------|--------------------|-----------------------------|----------------------------|
| ORDER CODE | PAEDIATRIC (BROWN) | SPECIAL | SPECIAL | SPECIAL | SPECIAL |
| | SMALL (RED) | SL1100 | SL1101 | SL1125 | SL1126 |
| | MEDIUM (YELLOW) | SL1103 | SL1104 | SL1128 | SL1129 |
| | LARGE (GREEN) | SL1106 | SL1107 | SL1131 | SL1132 |
| | EXTRA LARGE (BLUE) | SPECIAL | SPECIAL | SPECIAL | SPECIAL |

IF YOU REQUIRE SIDE SUSPENDERS, PLEASE ADD A 'C' TO PART NUMBER



Joerns Healthcare Limited
 Drakes Broughton Business Park, Worcester Road
 Drakes Broughton, Pershore, Worcestershire
 WR10 2AG United Kingdom
 Tel: 0844 811 1156 • Fax: 0844 811 1157
 info@joerns.co.uk • www.joerns.co.uk

