


# Oxford®

## QUICKFIT DELUXE SLING

### USER GUIDE FOR THE QUICKFIT DELUXE SLING

The Quickfit Deluxe Sling is a development of the Quickfit design to suit 95% of clients. It provides a more comfortable and more supportive lift without the need for side suspenders. It is available with padding and padding plus head support and it is available in five sizes - extra large, large, medium, small and paediatric.

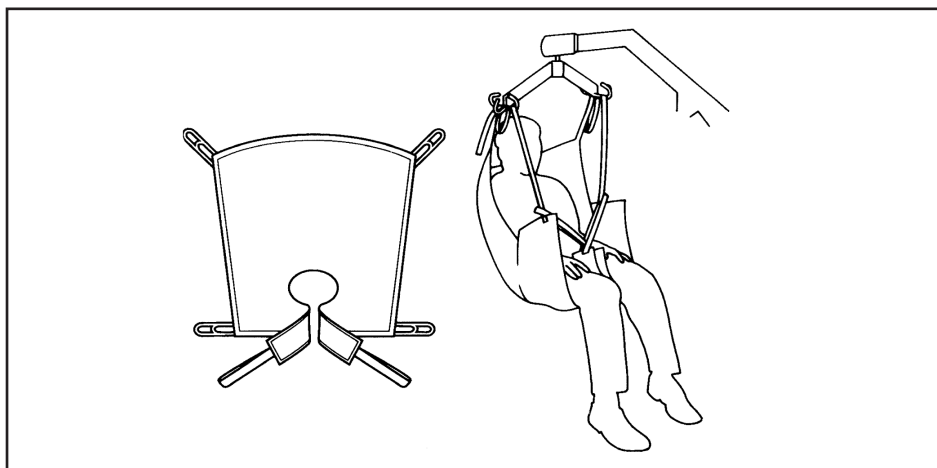

#### **OXFORD RECOMMENDS THE USE OF GENUINE OXFORD PARTS.**

*Oxford sling and lift products are designed to be compatible with one another. For country specific guidance on sling use and compatibility, please refer to the sling label or contact your local market distributor or Joerns Healthcare.*

*For the safety of the patient and carer, before using a sling a full risk assessment must be conducted to ensure that the correct sling choice, method of positioning in the sling and procedure for transfer has been determined for the patient.*

*Check sling and stitching before each use. Using bleached, torn, cut, frayed or broken slings is unsafe and could result in serious injury or death to the patient.*

*Destroy and discard worn slings. Do not alter slings.*

*Never leave a patient unattended.*




**ADDITIONAL SLINGS ARE AVAILABLE TO SUIT INDIVIDUAL NEEDS.  
YOU ARE ADVISED THAT YOU SHOULD ALWAYS SEEK THE ADVICE OF AN  
AUTHORISED DEALER BEFORE PURCHASING.**


**WE WILL BE PLEASED TO ADVISE YOU OF YOUR NEAREST AUTHORISED DEALER.**




## HOW TO FIT FROM SEATED POSITION

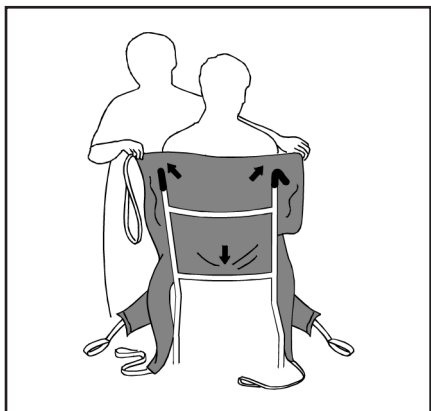


Make sure the hand grips and seams are on the outside of the sling. Feed the sling down the back of the patient, leaving the commode aperture at the base of the spine. Check the sling is square across the shoulders.

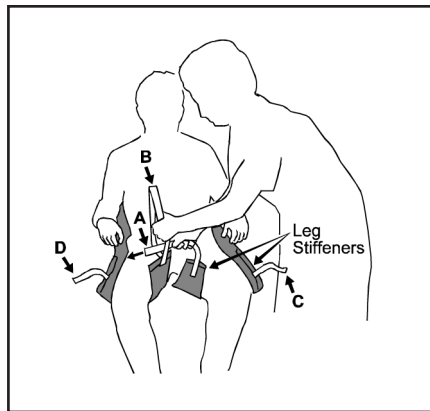

Bring the longest leg piece stiffener forward so that it is parallel with the long bone of the thigh. Ensuring that the leg piece is not twisted, take the shortest leg piece stiffener under and up between the legs. Repeat this procedure for the other leg. Cross strap A through strap B.

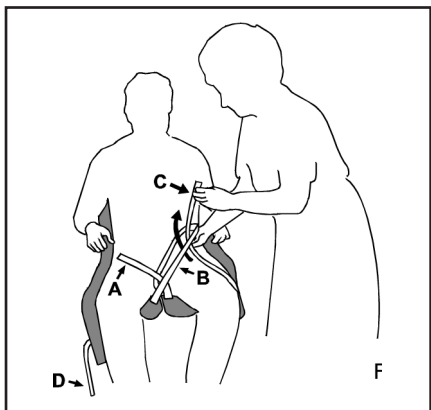


Pass strap C up through strap B then repeat for other leg using strap D through strap A.

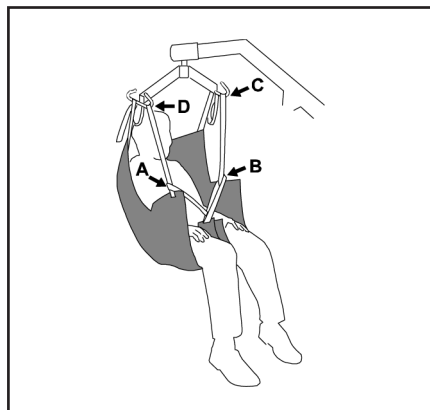


Check the sling is smooth under the patient and the position is comfortable. Move the hoist into position and attach the leg straps to the spreader bar using the coloured loops. This maintains the patient in an upright sitting position. The lift may move towards the patient as you do this. When reseating the patient the hand grips may be used to ensure a good posture.

## ALTERNATIVE SEATED POSITION

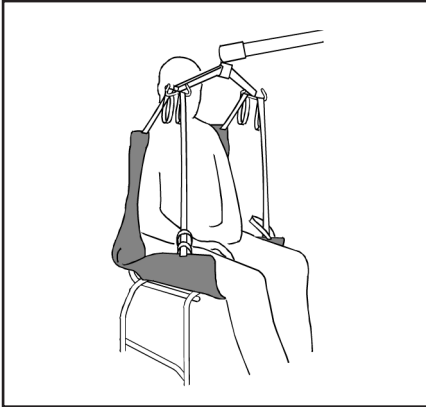

For closed leg positions, straps may be crossed under both legs and attached as shown. Ensure leg straps C and D are passed through straps A and B and attached to the spreader bar. This can provide additional comfort and can be used for some amputees.

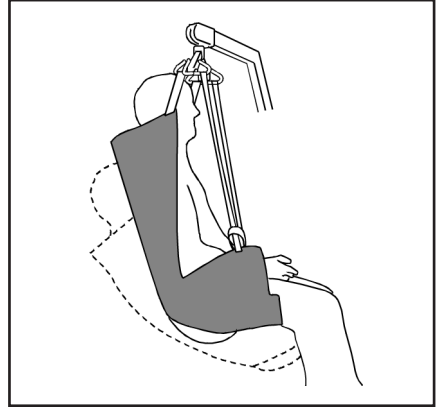


The longer you attach the shoulder straps, the more reclined the client will be.




**There is an increased risk of slipping out of the sling using this method of transfer. Assess the patient's capabilities carefully.**

## RECUMBENT POSITION

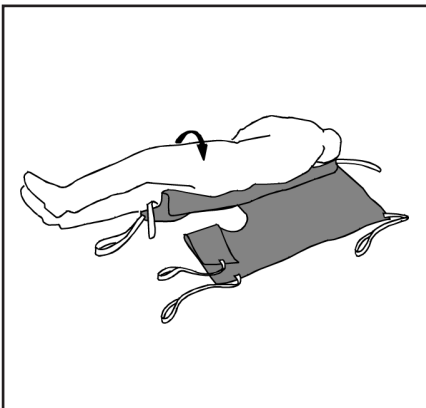

Draw sheet roll the client onto the sling, ensuring that the commode aperture is at the base of the spine.

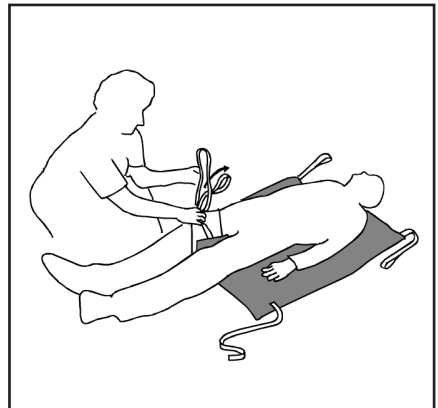

Cross leg strap A and B under and between the legs and feed A through B. Feed C and D through the long loops on A and B. Then proceed to attach the sling to the spreader bar in the same way as the sitting position.


Joerns Healthcare recommend slings are checked regularly and before use for fraying or damage.

## QUICKFIT DELUXE SLING TECHNICAL SPECIFICATIONS

STANDARD	BS EN ISO 10535
SAFE WORKING LOAD	500lbs / 227kg

## SIZING

Each sling is marked with the following colour coded symbols, four of which will be crossed. Your sling is the uncrossed size, i.e. in this example Small (Red).

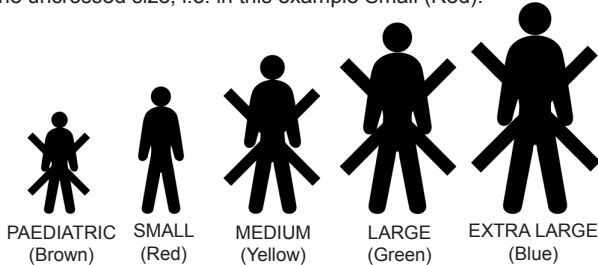

## WASHING INSTRUCTIONS

Each sling is marked with the following washing instructions:

**DO NOT WASH WITH BLEACH**

Bleach will damage material. Slings can suffer damage during washing and drying. Check sling before each use. Bleached, torn, cut, frayed or broken slings are unsafe and could result in serious injury or death to patient. Destroy and discard worn slings. Do not alter slings. Machine wash warm. Cool tumble dry, air dry or dry at very low temperature.



		Quickfit Deluxe (Polyester)	Quickfit Deluxe (Net)	Quickfit Deluxe Padded (Poly)	QF Deluxe Padded HS (Poly)
ORDER CODE	PAEDIATRIC (BROWN)	SPECIAL	SPECIAL	SPECIAL	SPECIAL
	SMALL (RED)	SL1050	SL1051	SL1050P	SL1060P
	MEDIUM (YELLOW)	SL1053	SL1054	SL1053P	SL1063P
	LARGE (GREEN)	SL1056	SL1057	SL1056P	SL1066P
	EXTRA LARGE (BLUE)	SPECIAL	SPECIAL	SPECIAL	SPECIAL



Joerns Healthcare Limited  
Drakes Broughton Business Park, Worcester Road  
Drakes Broughton, Pershore, Worcestershire  
WR10 2AG United Kingdom  
Tel: 0844 811 1156 • Fax: 0844 811 1157  
info@joerns.co.uk • www.joerns.co.uk

