


Oxford®

STANDING HARNESS

USER GUIDE FOR THE STANDING HARNESS

The Standing Harness is a versatile sling with a wide range of uses. This includes assisting with standing and walking or visiting the toilet. The Standing Harness supports the patient around the upper body as well as under the arms. The Standing Harness is not a general-purpose sling and requires the patient to be able to partially weight bear.

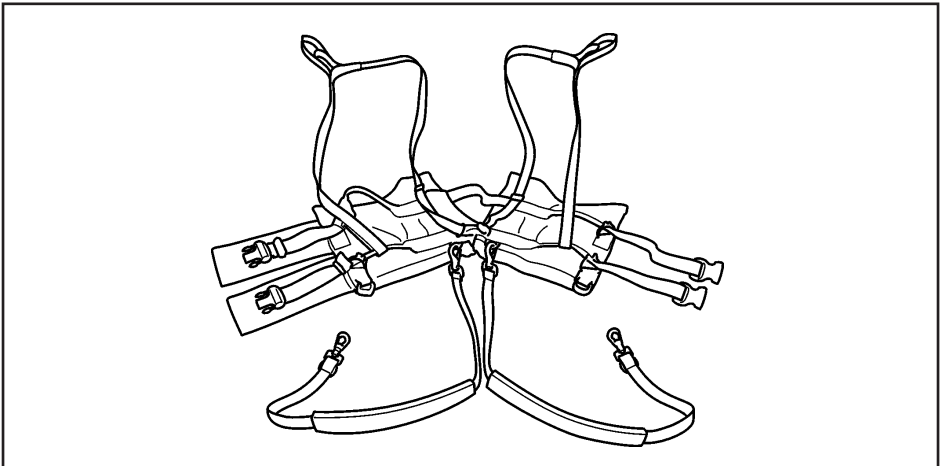

OXFORD RECOMMENDS THE USE OF GENUINE OXFORD PARTS.

Oxford sling and lift products are designed to be compatible with one another. For country specific guidance on sling use and compatibility, please refer to the sling label or contact your local market distributor or Joerns Healthcare.

For the safety of the patient and carer, before using a sling a full risk assessment must be conducted to ensure that the correct sling choice, method of positioning in the sling and procedure for transfer has been determined for the patient.

Check sling and stitching before each use. Using bleached, torn, cut, frayed or broken slings is unsafe and could result in serious injury or death to the patient.

Destroy and discard worn slings. Do not alter slings.

Never leave a patient unattended.




ADDITIONAL SLINGS ARE AVAILABLE TO SUIT INDIVIDUAL NEEDS.
YOU ARE ADVISED THAT YOU SHOULD ALWAYS SEEK THE ADVICE OF AN
AUTHORISED DEALER BEFORE PURCHASING.

WE WILL BE PLEASED TO ADVISE YOU OF YOUR NEAREST AUTHORISED DEALER.


HOW TO FIT THE STANDING HARNESS

Method of attachment from a chair:


NOTE: Prior to fitting, a thorough risk assessment should be conducted.

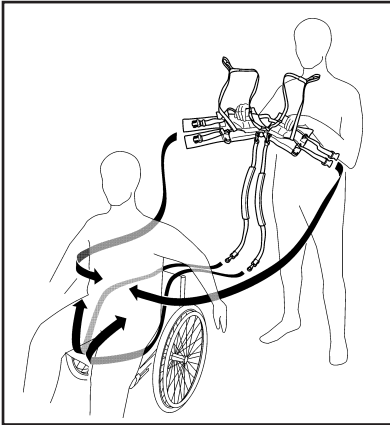


Feed the Standing Harness down the back of the patient and position around the chest.

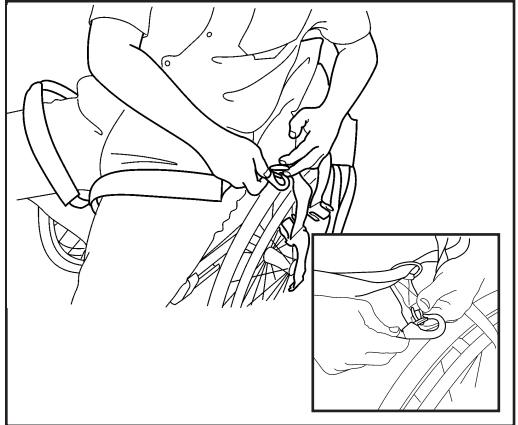

Attach the leg supports to the Standing Harness by offering the safety clasp to the retaining ring at the bottom of the Standing Harness as shown. The leg supports can be adjusted in length for comfort as appropriate.

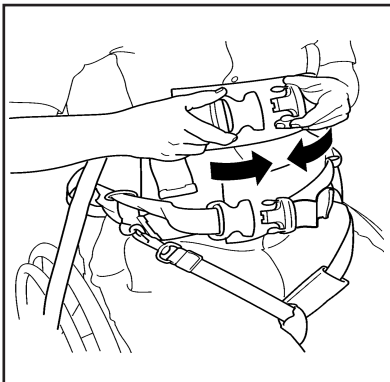

Raise and position the Standing Harness up towards the chest. The Standing Harness is equipped with two adjustable safety clasps. Attach both safety clasps as shown and tighten for comfort as appropriate (the fit should be snug not tight).

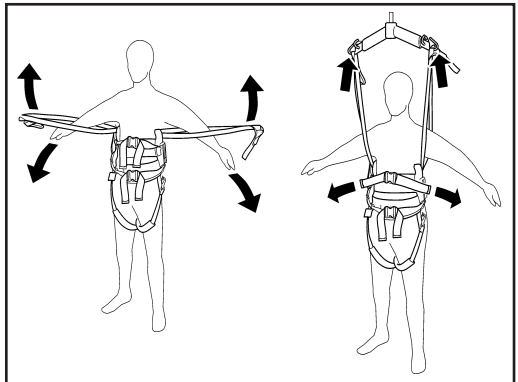


To attach the Standing Harness to the spreader bar, firstly place the patient's arms through the loop straps as shown, and attach to the spreader bar by using the coloured loop straps. Ensure the same colour loop is used on both straps and hook onto the side hooks. Check both loops are securely attached prior to use.


HOW TO FIT THE STANDING HARNESS

Method of attachment for more able-bodied patients:


NOTE: Prior to fitting, a thorough risk assessment should be conducted.

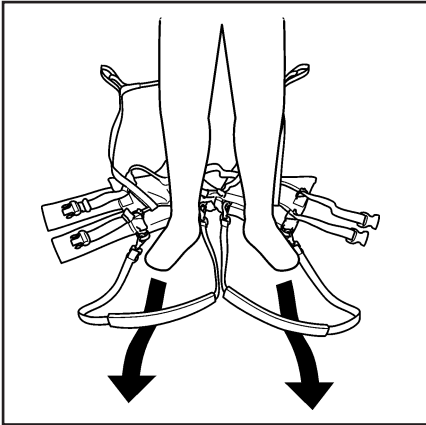

Position the patient's legs into the leg supports. You can adjust the length of the leg supports for comfort as appropriate.

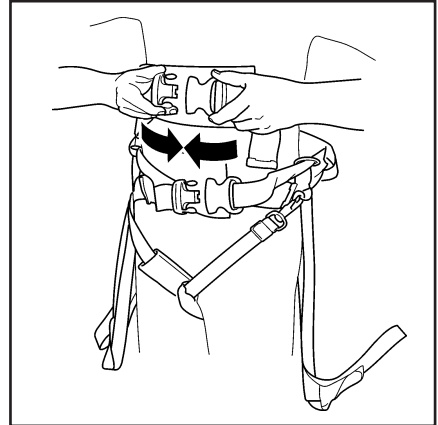

Raise and position the Standing Harness up towards the chest. The Standing Harness is equipped with two adjustable safety clasps. Attach both safety clasps as shown and tighten for comfort as appropriate (the fit should be snug, not tight).

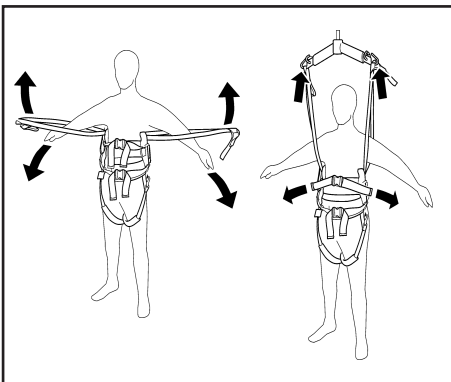

To attach the Standing Harness to the spreader bar, firstly place the patient's arms through the loop straps as shown, and attach to the spreader bar by using the coloured loop straps. Ensure the same colour loop is used on both straps and hook onto the side hooks. Check both loops are securely attached prior to use.

ATTENTION:


- Check that the safety clasps and the connection loops are all securely attached prior to using the Standing Harness.
- The Standing Harness is not a general-purpose sling and will not be suitable for some clients.
- The Standing Harness is only suitable for the Oxford/Hoyer 6 point spreader bar.

STANDING HARNESS TECHNICAL SPECIFICATIONS

STANDARD	BS EN ISO 10535
SAFE WORKING LOAD	500lbs / 227kg

SIZING

Each sling is marked with the following colour coded symbols, four of which will be crossed. Your sling is the uncrossed size, i.e. in this example Small (Red).

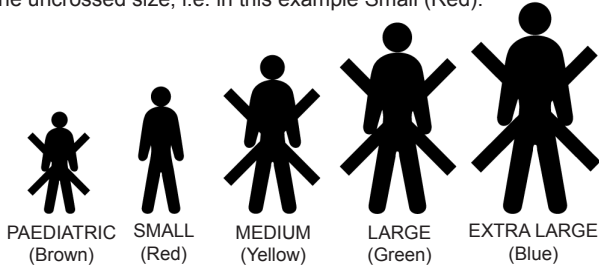

WASHING INSTRUCTIONS

Each sling is marked with the following washing instructions:

DO NOT WASH WITH BLEACH

Bleach will damage material. Slings can suffer damage during washing and drying. Check sling before each use. Bleached, torn, cut, frayed or broken slings are unsafe and could result in serious injury or death to patient. Destroy and discard worn slings. Do not alter slings. Machine wash warm. Cool tumble dry, air dry or dry at very low temperature.



		Standing Harness (Polyester)
ORDER CODE	PAEDIATRIC (BROWN)	SL26600
	SMALL (RED)	SL26601
	MEDIUM (YELLOW)	SL26602
	LARGE (GREEN)	SL26603
	EXTRA LARGE (BLUE)	SPECIAL

THESE SLINGS ARE SUITABLE FOR USE WITH THE OXFORD 6 POINT SPREADER BAR.



Joerns Healthcare Limited
 Drakes Broughton Business Park, Worcester Road
 Drakes Broughton, Pershore, Worcestershire
 WR10 2AG United Kingdom
 Tel: 0844 811 1156 • Fax: 0844 811 1157
 info@joerns.co.uk • www.joerns.co.uk

