Oxford Slings

Additional Important Safety Information

Following Medical Device Alerts on other manufacturers' slings, it is important that users of Oxford Slings always follow the instructions in the User Manuals provided with each sling and also the labels which are stitched into each Oxford sling.


In particular, it is critical to the safety of the patient that the sling is checked each and every time prior to use. Failure to do so may result in patient injury.

If any of the following faults are identified, the sling should not to be used and destroyed:

- There are signs of general wear and tear.
- There is loose or broken stitching on any part of the sling (including the straps)
- There are worn areas on the attachment points.
- There are discoloured areas
- There are areas of frayed or torn material
- There are areas of bleached material
- The label is illegible

Inspection is required on all areas of the sling. The following information provides some guidance on key areas for inspection:

Example of an Oxford Sling:

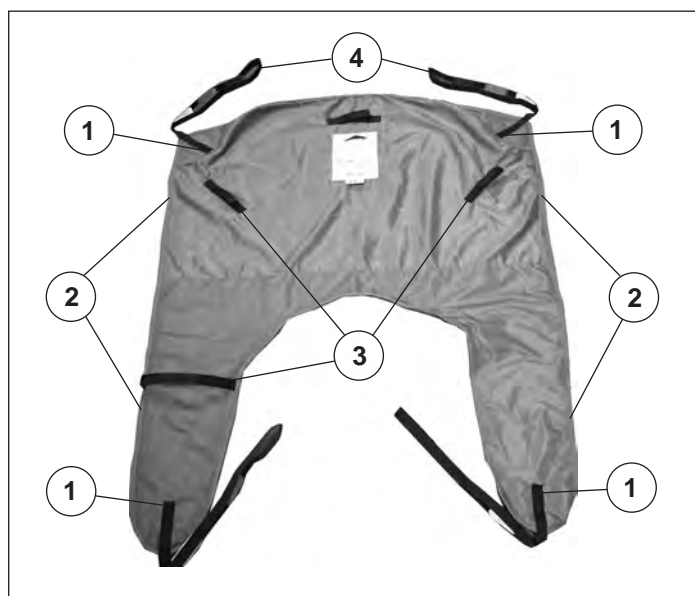Key areas for inspection include:

1. Sling strap attachment point
2. Seams
3. Handle attachment points
4. End loops on sling straps


Important Safety Information

Issued March 2010


Oxford®
Slings



- Stitching on the sling straps on the sling body should be secure.
- Stitching is cross-stitched and not to be frayed.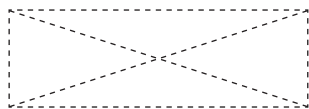


Cross Stitching

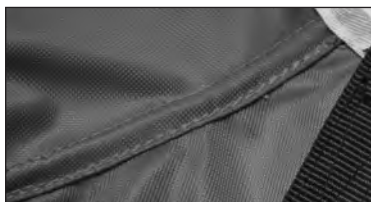- Typical seam stitching.
- Photograph shows example of correct stitching in place and not frayed.

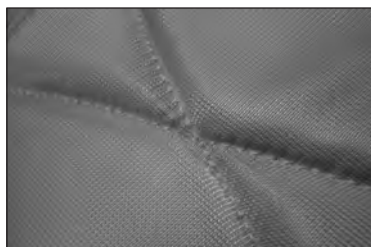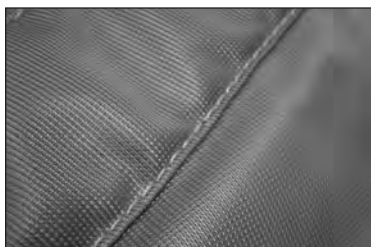
- Typical stitching on the main body of the sling.
- Photographs show examples of correct stitching in place and not frayed.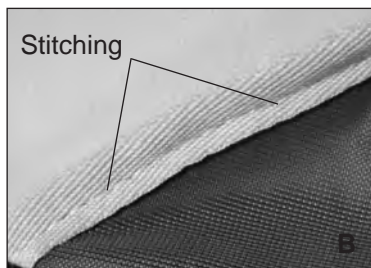

Stitching


- A. Attachment of the strap to a net style sling.
- B. Example of correct stitching on the colour binding.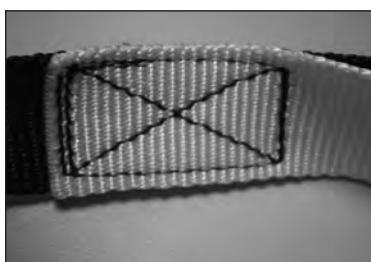

- Typical stitching of the end loops on the sling straps.
- Stitching is cross-stitched and not to be frayed.
- Ensure end loops are not worn or frayed.

Cleaning Instructions are clearly detailed in the User Manual and on the label stitched on the back of the sling. Failure to follow these instructions may result in the sling becoming damaged and unsafe for use.

If you have any questions regarding these instructions please contact Oxford Customer Services on +44(0)844 811 1156.

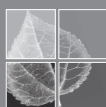Joerns Healthcare Limited
Drakes Broughton Business Park, Worcester Road, Drakes Broughton
Pershore, Worcestershire, WR10 2AG United Kingdom
(T) +44 (0) 844 811 1156 (F) +44 (0) 844 811 1157
www.joerns.co.uk • email: info@joerns.co.uk

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